



TOILET SEAT LIFT

INSTRUCTION MANUAL

www.elevaclic.es

Battery DL2



TOILET SEAT LIFT

INSTRUCTION MANUAL

To ensure proper use of this appliance and for your safety, please read this manual completely before installation and use.




DIMENSIONS

Precautions before installation:

- First measure the width of the toilet, which must be less than 45 cm-17".
- The space on both sides of the toilet must be at least 12 cm-6".


- The angle of elevation is 30°; this area must be free of objects and obstacles.

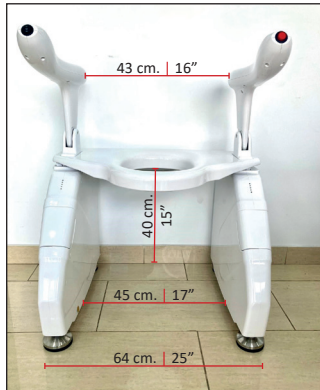

ACCESSORIES



A - Handrail locking screws x2

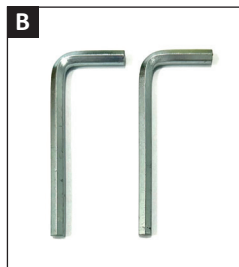

B- Handrail Allen wrench x2



C- Remote control x1



D - Toilet mounting screws x2



E - Toilet mounting hex key x1



F - Charger x1

SPECIFICATIONS

Product specifications:

- Power supply for charging batteries: **220-240V AC, 50-60Hz**.
- Operating voltage: **24V**.
- Rated power: **130W**.
- Batteries can be charged **120 times** for raising and lowering the toilet on a single charge.
- Waterproof rating: **IP44**.
- Tilt angle: **30°**.
- Installation requirements: Toilet width: **<45 cm <17"**.
- Maximum load: **220 Kg**.
- Product weight: **25 kg**.


CLEANING AND MAINTENANCE

- The surface of this product is made of high-strength engineering plastic and can be cleaned with a damp cloth.
- Do not use abrasive cleaning products or rinse with water.
- The main unit requires no maintenance or lubrication.
- Avoid getting the main unit's push bar wet.
- If used by multiple people, disinfection after each use is recommended.


PRODUCT FUNCTION



One-click lift button,
optional position right or left.

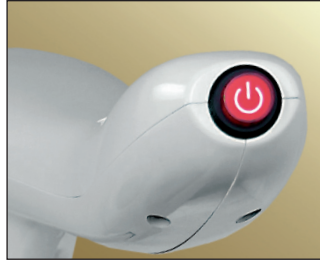

One-click alarm button,
optional position right or left.

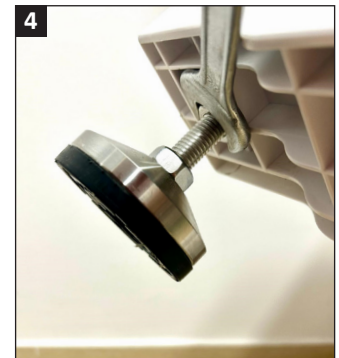


ASSEMBLY AND ADJUSTMENT OF THE LEGS

1. The legs can be raised and lowered from 0 to 15 cm - 6".
2. Adjust the height of the lift legs so that the rear crossbeam matches and adjusts to the height of your toilet and the lift is level in the lowered position.
3. Once you have the desired height and correct leveling, turn and raise the nut until it is positioned at the top of the leg.
4. Gently tighten the nut with a flat wrench until it is secure and firmly in place.

ARMREST INSTALLATION AND ASSEMBLY

Connect the arm's power cable to the frame's power cable. Align the connection and tighten the collar, ensuring the connection matches, as shown in the image.

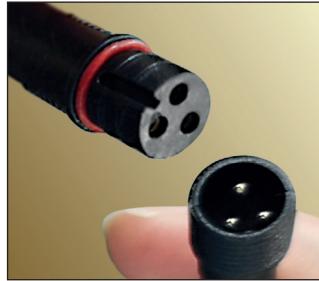

Align the armrest's orientation as shown in the image **(A)** and then push it into the groove, locking it down **(B)**. Next, secure the armrest to the main frame body with the locking screws **(C)**. Tighten with the Allen wrench **(D)**.



ATTACHING THE FRAME TO THE TOILET

1. Unscrew the toilet seat and lid, leaving only the ceramic base.
2. Insert the black rubber dowels into the two holes in the toilet bowl.
3. Remove the white plastic protective cover from the frame's crossbeam. To remove it, pull the plastic cover vertically. Secure the dowels to the flat metal piece with the long bolt. Slide the frame towards the toilet bowl.
4. Align the metal plate with the hole in the toilet bowl. Using the Allen wrench and two screws, secure the crossbeam to the flat metal brackets attached to the dowels.



Remember: Replace the crossbeam's protective cover in its original position.



ABOUT BATTERIES




Charging methods:

This product is equipped with two high-capacity batteries and operates independently; it does not require an external power source.

1. The blue light is on when fully charged. The number of green lights indicates the remaining battery power. When the red light is on, it needs charging; charge it as soon as possible.

2. When the remote control is not in use for an extended period, turn off the receiver switch. Keeping it on will continuously consume power.

Pay attention to reasonable use to ensure a long battery life.

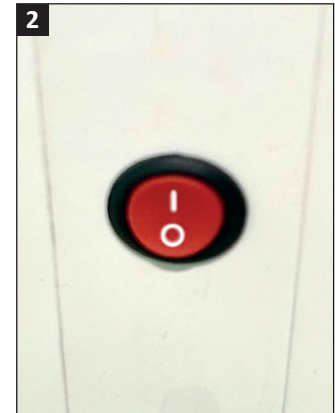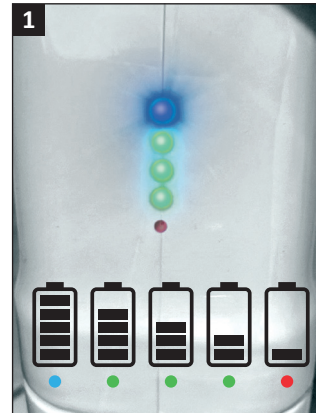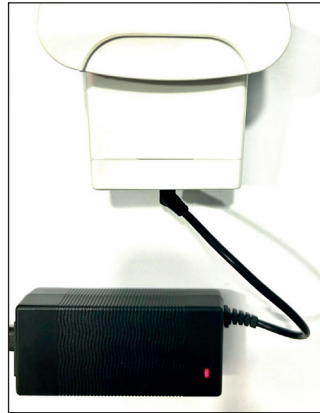

NOTES, WARNINGS, AND PRECAUTIONS

- 1- Carefully read the instructions in this manual completely.
- 2- Users with limited physical capabilities require supervision or assistance when using this product.
- 3- Before use, confirm that all components are properly assembled and tightened.
- 4- Do not exceed the product's maximum load capacity of 220 Kg.
- 5- The handrail is designed for easy gripping while standing or sitting; use both sides simultaneously.
- 6- Do not rest your entire body weight on a single armrest; distribute it evenly on both sides, with each side supporting half your weight.
- 7- Overloading the armrest may damage it and cause it to break.
- 8- Do not stand on the product.
- 9- Keep it away from heat sources.
- 10- Keep the batteries charged to at least 30% for proper operation.
- 11- Any issues related to this product should be reported to the distributor.
- 12- Environment: When this appliance reaches the end of its useful life, it should not be considered normal waste, but hazardous and polluting waste, as such, it must be deposited in the specific recycling and disposal areas, according to directive 2024/884/EU.



WARRANTY AND WARRANTY EXCLUSIONS

Warranty:

- The manufacturer guarantees that this product is supplied free from defects.
- All products are covered by the manufacturer's official warranty for a period of 3 years, in accordance with current regulations. The battery has a 2-year warranty.
- To make use of the warranty, you will need to provide the purchase invoice.
- In case of a major malfunction within 1 year of the purchase date, we will replace the product at no cost to the customer; we will not repair it. For after-sales support, please contact us info@elevaclic.es



Warranty exclusions:

1. Replacement parts not supplied by the manufacturer or authorized distributor. If any parts are missing or defective, please contact us.
2. Unit failures not caused by material or manufacturing defects, such as those resulting from misuse or damage caused by improper installation.
3. Natural wear and tear, regular parts replacement, breakage, or damage caused by external factors.
4. Damage caused by failure to follow the instructions for use or installation instructions.
5. The manufacturer and distributor are not liable for accidental damage, property damage, or material losses.
6. The warranty conditions are clear and explicit, guaranteeing the rights and interests of consumers and offering a solid guarantee for every customer.

 English



DISTRIBUTED BY: AZULCOR GESTION, S.L.U.

C.P. 41520 - EL VISO DEL ALCOR, SEVILLA
SPAIN

TLF. +34 649641255

info@elevaclic.es / www.elevaclic.es

Made in P.R. China

The distributor reserves the right to make modifications and improvements to its product, as well as slight color variations. No liability is accepted for printing or translation errors.